
LiNA LapGuard™

Sterile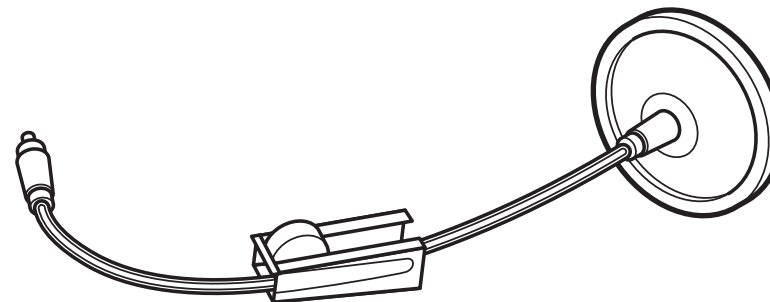

en

Intended Use:

LiNA LapGuard™ is a passive filter attached to a trocar with outlet. The filter removes odours and particles from the air escaping the pneumoperitoneum.

Patient Population:

All groups dedicated to laparoscopic electro-surgery.

Product description:

LiNA LapGuard™ is a laparoscopic smoke filtration system. With ULPA filter with active carbon, product offers evacuation and filtration of electro-surgical smoke during laparoscopic procedures.

Precautions:

- Carefully inspect the device for any damage prior to use. Dispose of the device if bent, malformed or if breaks or cracks are found.
- Do not use past expiration date.
- The LiNA LapGuard™ is provided sterile. Carefully inspect the packaging for any damage prior to use. Do not attempt to use the device if sterile barrier is damaged.
- The LiNA LapGuard™ should only be used by qualified OR/hospital staff with experience in the use of similar devices.

Warnings:

- No modifications of this equipment is allowed.
- For single use only. Do not reuse, reprocess or re-sterilize this device. Any reprocessing may impede the functions of this device. Reusing single use devices may also increase the risk of cross contamination. Attempts to clean the device results in risk of device malfunction.
- After use, dispose of product and packing in accordance with hospital, administrative and/or local government policy.

Instruction for use:

1. Open sterile package and attach LiNA LapGuard™ connector to the cannula.
2. Place and clamp filter away from the operative site.
3. The roller clamp may be adjusted to set the desired flow of smoke.

Reporting:

Any serious incident that has occurred in relation to the device should be reported to the LiNA Medical ApS and the competent regulatory authority of the country in which the user and/or patient is established.

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